Vital info when seconds matter

User manual | For rescue professionals



What is twlCEme?

twlCEme is a digital device that makes vital info easily accessible in emergency situations by using your phone. The twlCEme symbol on sports or work equipment indicates an integrated device. The user can upload personal information such as:












Medical info

ICE contacts

Personal identifiers

Exact coordinates

Insurance info

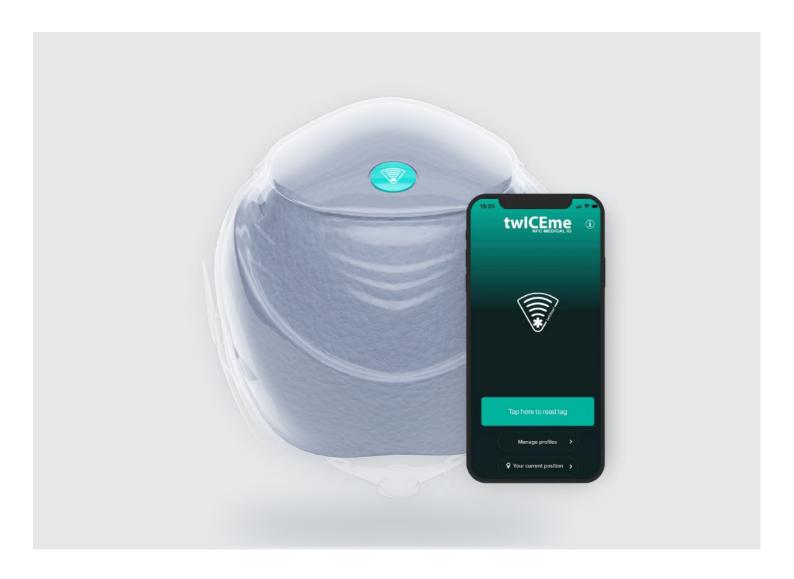

Get started

Download the free app from <u>App Store</u>, <u>Google Play</u> or by scanning the QR code. twlCEme can be used by all NFC enabled smartphones.





How to use twlCEme

in case of emergency

1

Find the symbol



Arrival at an emergency scene: while doing patient assessment, look for the twlCEme symbol on personal equipment of an injured person.

2

Place NFC antenna







Place NFC antenna of your phone on the twlCEme symbol within max.1 cm distance from the equipment.

Make sure to know the placement of an NFC antenna on your phone and that NFC is turned on.

3

Scan to access information



In the twlCEme app, press 'Tap here to read tag'. Information is also accessible without the app by using the background scan functionality.

4

Read information



Vital information of the person in question will appear on the screen of your smartphone. The information is presented in order of relevance.

Equipment with twlCEme

twlCEme is integrated into equipment like helmets, jackets, backpacks or full body harnesses in the bike, snow and construction industry. twlCEme products are now available on market with growing numbers each day.

Here are some examples of equipment with twlCEme.

Integrated into equipment











Construction helmets



Bike helmets



Snow helmets



Fall protection harnesses



Complete product range

The number of products with twlCEme is growing every day. To see the latest product range, visit our website.

twiceme.com/equipment

Free information kit

Try twlCEme for yourself – order a free information kit. That way you will be best prepared in case of an emergency. Contact us via e-mail or order a kit by filling in the form on our website. We might contact you prior sending out the kit in case we have some extra questions.

info@twiceme.com

twiceme.com



Spread the word

Inform your colleagues about twlCEme. twlCEme works best when as many people as possible know about it. Together we can spread the knowledge of accessible vital information faster.







Our constantly growing rescue community

We welcome feedback

twiceme.com

Industrigatan 4C, SE-112 46 Stockholm

info@twiceme.com